

### Traditional Pregnancy Test



### Owner's Booklet

Manufactured For: GENESIS BIOTECH PTY LTD  
Address: Level 1, 16 McDougall Street,  
MILTON QLD 4064. [www.mylifemart.net.au](http://www.mylifemart.net.au)

**TaiDoc Technology Corporation**  
3F., 5F., No. 127, Wugong 2nd Road, 24888  
Wugu Dist., New Taipei City, Taiwan  
**MedNet GmbH**  
Borkstraße 10, 48163 Münster, Germany

### Symbol Information

Symbol	Referent
	For in vitro diagnostic use only
	Read instructions before use
	Temperature limitation
	Use by/ Expiry date
	Batch code
	Manufacturer
	Authorised representative in the European Community

Genesis Biotech Pty Ltd.  
Level 1, 16 McDougall Street,  
MILTON QLD 4064  
[www.mylifemart.net.au](http://www.mylifemart.net.au)  
**CONTACT US**  
For Australia & New Zealand customers  
[support@mylifemart.net.au](mailto:support@mylifemart.net.au)  
Australian customers 1300 110 155

2°C - 30°C 0123  
For self-testing  
Version 2.0 2011/11 312-5302103-002

### Introduction

The LS-633 Traditional Pregnancy Test is intended to determine human chorionic gonadotropin (hCG) concentration in a urine sample through the specific interaction between antibodies and antigens. It is over 99% accurate in detecting the pregnancy hormone and provides a rapid, easy-to-use test within 3-5 minutes after applying the urine sample.


This package contains Traditional Pregnancy Test(s) and your owner's booklet.

### PLEASE READ INSTRUCTIONS CAREFULLY BEFORE USE


- For *in vitro* diagnostic use only.
- For self-testing.
- Store at 2°C - 30°C (35.6°F - 86°F). Do not freeze.
- Use this test at the room temperature.
- Use the test immediately after opening the foil wrapper.
- Keep dry and away from sunlight.
- Keep out of the reach of children.
- Do not reuse. Dispose of properly after use.
- Do not use the test after the expiration date printed on the package.
- Do not use the urine that contains blood.
- This product only provides a preliminary test for pregnancy. You should contact your doctor for further diagnosis.

### Start Testing

#### Step 1 Remove the test stick from the foil wrapper

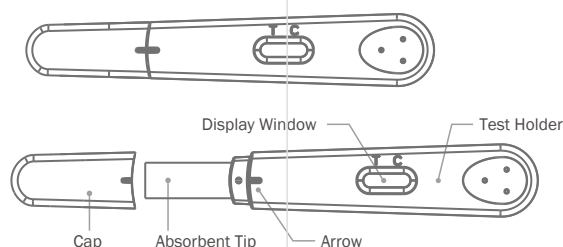

#### Step 2 Take off the cap

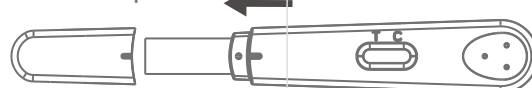



#### Step 3 Perform the test

Use a clean plastic or glass container to collect a urine sample. Hold the test holder in a vertical position and immerse the entire absorbent tip into the sample for 10-15 seconds.

**OR**

Place the absorbent tip pointing downward and face up in your urine stream for 5-7 seconds.

- Any urine sample can be used for hCG testing. The first urine of the morning is ideal because it generally contains the highest concentration of hCG.
- Please perform the test immediately after urine collection.
- Do not use the urine sample if the urine appears turbid or contains unknown matter.

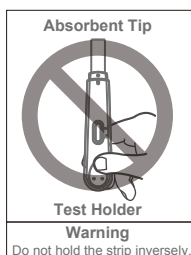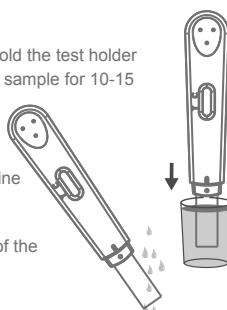


#### Step 4 Wait for 3-5 minutes

After applying your urine, cover the strip with the cap and lay the strip flat facing up on a clean, dry, non-absorbent surface. Do not hold the test stick with the absorbent tip pointing upward.

#### Step 5 Result appears

Red color line(s) is in the display window.

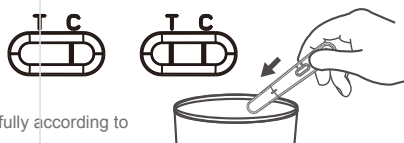

Discard the used test stick with the cap carefully according to your local applicable regulations.

### FURTHER INFORMATION

#### Result Interpretation



**TWO RED LINES** in the display window:  
This indicates that you may be pregnant. You should see your doctor who can advise you on what steps you should take next.



**ONE RED LINE** in the display window:  
This indicates that you may not be pregnant, or the level of pregnancy hormone may not yet be high enough to be detected, or you may have miscalculated the day your period is due.

### Test Errors

Error Message	Possible Cause	What to Do
	► Too much or too little urine was applied. ► The strip was not laid flat after urine was applied. ► Other problem in operation.	► Check the expiration date of the test stick. ► Test again with a new test stick and follow the instructions carefully. If the display remains blank, please contact local customer services for assistance.

### Warning!

- False-negative results may occur when the level of pregnancy hormone in your urine is not high enough to be detected. If pregnancy is still suspected, you should collect the first urine of the day and repeat the test again. Please remember to avoid excessive fluid intake before testing.
- Some diseases or cancers may affect the result by interfering with pregnancy hormone production. If you have the above conditions, please contact your doctor before performing a Traditional Pregnancy Test.
- Medications containing hCG may affect the test result and should not be taken while using this test. Consult your physician before using this test.
- If you have no idea when your period is due, we recommend testing at least 19 days after the last time you had unprotected sex.
- If you are in or approaching the menopause, you may receive a false-positive result.
- Ectopic pregnancy and ovarian cysts may cause misleading results.
- To obtain accurate results, please follow the procedures precisely.
- The maximum time of interpreting your result is 5 minutes after it first appears.

### Accuracy

The LS-633 Traditional Pregnancy Test is over 99% accurate in detecting the pregnancy hormone in laboratory results when using 209 urine samples from pregnant and non-pregnant women. The pregnancy result can be detected one day after missing the menstruation.

### Specification

Model no.: LS-633  
Dimensions & Weight: 131 mm(L) x 23.2 mm(W) x 8.3 mm(H), 11.5g

Storage conditions: 2°C - 30°C (35.6°F - 86°F), below 85% R.H.  
Operating Temperature: Allow to reach room temperature for 30 minutes prior to testing if stored refrigerated.